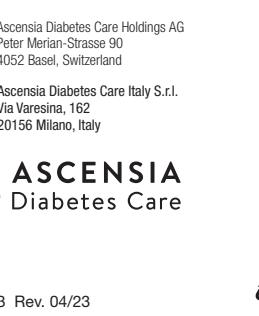
UKRP (UK Responsible Person): Ascensia Diabetes Care UK Ltd
Ascensia House, Albert Road, Newbury, Berkshire RG14 1DL UK
Ascensia Diabetes Support 0345 600 6030 - UK
Imported and Distributed by: Ascensia Diabetes Care Ireland Ltd, 10 Earlsfort Terrace, Dublin 2, D02 T380, Ireland
Ascensia Diabetes Support 1 800 920 111 - IE
www.diabetes.ascensia.com
Ascensia Diabetes Care Holdings AG Peter Merian-Strasse 90 4052 Basel, Switzerland
EC REP Ascensia Diabetes Care Italy S.r.l. Via Varesina, 162 20156 Milano, Italy

90010353 Rev. 04/23

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INTENDED USE: Intended for collecting capillary blood, for use together with disposable lancets. The **MICROLET NEXT lancing device** is used to draw capillary blood from the fingertip or palm for blood glucose testing with compatible lancets. The **MICROLET NEXT lancing device** is for single-patient use. The lancing device is not intended for neonates. The **MICROLET lancet** is a single-use sterile device intended for drawing a capillary blood sample from an adult or child for glucose monitoring. The lancet is not intended for neonatal use. There are no known contraindications associated with these devices.

INSTRUCTIONAL INSERT

- Lancet ejector for safe lancet disposal.
- Black/white adjustable endcap for fingertip testing only.
- Clear AST endcap for alternative site testing only.

Use with MICROLET® lancets for fingertip and palm testing.

Do not use AST under the following conditions:

- If you think your blood glucose is low (hypoglycemia).
- When blood glucose is changing rapidly (after a meal, insulin dose, or exercise).
- If you are unable to feel symptoms of low blood glucose (hypoglycaemic unawareness).
- If you get alternative site blood glucose results that do not agree with how you feel.
- During illness or times of stress.
- If you will be driving a car or operating machinery.

WARNING

- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

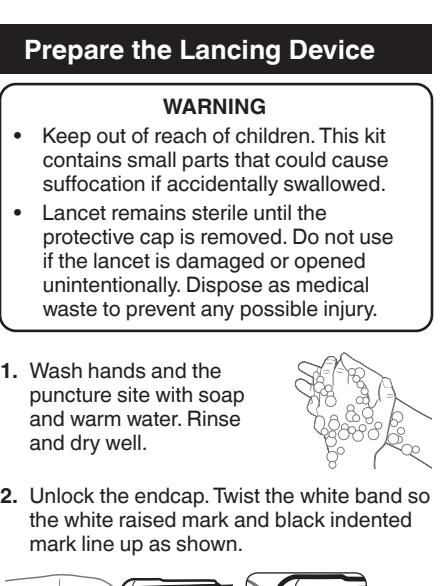
WARNING: Potential Biohazard

- The MICROLET NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.

CAUTION:

- Contact Customer Service for assistance if your device malfunctions for any reason or if you notice any difficulties using the device.
- Choose alternate puncture sites to prevent excess scarring, which could lead to loss of sensitivity.
- Test your blood glucose level as frequently as recommended by your healthcare provider.*

*Healthcare agencies recommend that adult patients test their BG three to four times per day (2018 U Ojha et al., "Disruption in the diabetic device care market") and that children and young persons test at least five times per day (2016 NICE [NG18], "Diabetes (type 1 and 2) in children and young people: diagnosis and management").


*May not be included in all kits. Available on request from Customer Service.

Prepare the Lancing Device


WARNING

- Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.
- Lancet remains sterile until the protective cap is removed. Do not use if the lancet is damaged or opened unintentionally. Dispose as medical waste to prevent any possible injury.

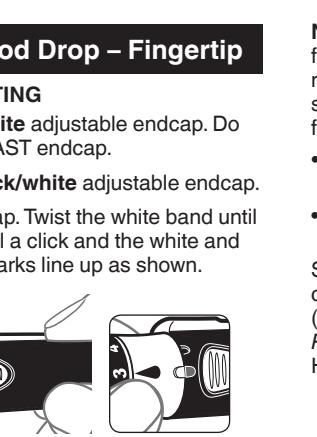3. Pull off the endcap.
4. Loosen the round protective cap on the lancet by rotating the cap ¼ turn, but do not remove the cap.

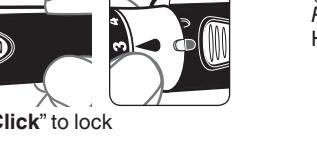5. Insert the lancet firmly into the lancing device. Push until the lancet comes to a complete stop.

"Click" to lock

6. Carefully twist off the round protective cap. Save the cap for disposing of the used lancet.

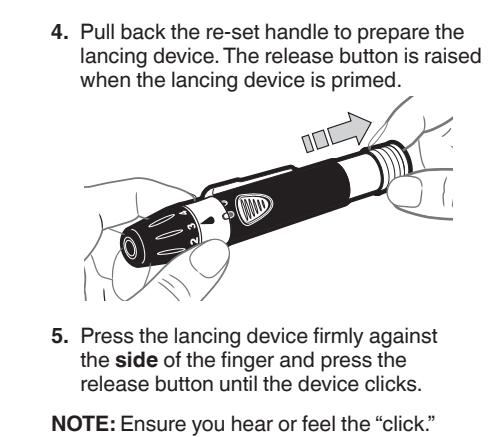
"Click" to unlock

3. Rotate the endcap dial to adjust the puncture depth. The amount of pressure applied to the puncture site also affects puncture depth.

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5. Press the lancing device firmly against the side of the finger and press the release button until the device clicks.

"Click"

If the device does not puncture, re-set by repeating Step 4.

Continued on reverse side ➔

NOTE: The Capillary Sampling guidelines from the World Health Organization (WHO) recommend a depth for pediatric finger-prick sampling using a lancing device / lancet as follows:

- For a child over 6 months to 8 years: MICROLET NEXT Setting 1 or 2
- For a child over 8 years: MICROLET NEXT Setting 4

Settings should be adjusted based on comfort. Pressure applied can affect depth. (*WHO Guidelines on Drawing Blood: Best Practices in Phlebotomy*, Geneva: World Health Organization; 2010.)

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Eject / Dispose of the Lancet

WARNING

Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.**

"Click" to unlock

2. Place the round protective cap that you saved onto a flat surface. With the used lancet still in the lancing device, push the lancet into the center of the cap.

5. Attach and lock the endcap. Twist the white band to lock.

3. Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.

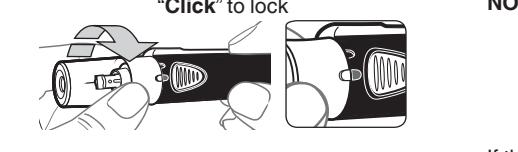
Get the Blood Drop – Palm

ALTERNATIVE SITE TESTING (AST) PALM

NOTE: Use the clear AST endcap.

1. Insert the lancet as described in *Prepare the Lancing Device*.
2. Attach and lock the clear AST endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

4. Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back.

"Click" to lock

CAUTION: Do not use the clear AST endcap on your fingertips.

3. Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.

4. Pull back the re-set handle to prepare the lancing device. The release button is raised when the lancing device is ready.

5. Press the lancing device firmly against the puncture site, then press the release button until the device clicks.

7. Lift the lancing device straight up and away from the skin without smearing the blood drop.

See *Eject / Dispose of the Lancet*.

Cleaning and Disinfection

Clean and disinfect your lancing device once a week. Use cloths dampened with 70% isopropyl alcohol.

CAUTION: Do not use bleach (sodium hypochlorite).

1. Clean the lancing device with 70% isopropyl alcohol and disinfect the lancing device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.

2. Allow to air dry.
3. Remove the endcap and eject the lancet, if present.
4. Get a new cloth dampened with 70% isopropyl alcohol and disinfect the lancing device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.
5. Allow the lancing device body and endcap to air dry.

Customer Service

Warranty: Your lancing device is warranted for 5 years.

Compatible lancets are MICROLET lancets.

If a serious incident has occurred during the use of this device or as a result of its use, please report it to the manufacturer and/or its authorized representative and to your national authority.

If you need assistance or have questions about blood glucose testing or your lancing device, contact Customer Service at 0345 600 6030 - UK , 1 890 920 111 - IE. For Customer Service Hours of Operation, visit www.diabetes.ascensia.com.

Symbols

LANCING DEVICE = Lancing Device

LANCES = Lancets

MD = Medical Device

AST = Alternative Site Testing (AST)

1 = Single patient, multiple use

⚠ = Caution

OTY = Quantity

Keep dry

UDI = Unique device identifier

Single sterile barrier system with protective packaging outside

Do not use if package is damaged

Low-density polyethylene*

Non-corrugated fiberboard (paperboard)*

Paper*

*Separate waste collection. Check your local municipal guidelines.

MICROLET NEXT lancing device made in Poland.

Ascensia, the Ascensia Diabetes Care logo, and Microlet are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

All other trademarks are properties of their respective owners. No relationship or endorsement should be inferred or implied.

© 2023 Ascensia Diabetes Care Holdings AG. All rights reserved.

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